


Product information: 20505

fully virucidal disinfectant

- included in the RKI disinfection list (Robert Koch Institut) acc. to §18 of the German Infection Protection Act (IfSG) for efficacy area AB
- usable for instrument disinfection, surface disinfection and linen disinfection in case of an officially ordered decontamination
- wide range of efficacy (incl. enveloped and non-enveloped viruses)
- Formalin > 35% DAB (German Pharmacopoeia)

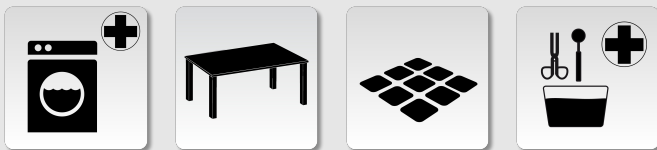

RHEOSEPT-D+ ultra

Disinfectant for use in case of an officially ordered decontamination

Product description

Fully virucidal disinfectant concentrate for the disinfection of instruments, linen and surfaces. For use in case of an officially ordered decontamination (disinfection) (acc. to §18 of the German Infection Protection Act [IfSG]) Listed with RKI (Robert Koch Institut) for efficacy area AB under the name Formaldehyde solution DAB 10 (formalin). Use biocides safely. Prior to use, please pay careful attention to the labelling and read the product information.

Fields of application

For prevention and control of infectious diseases. For institutional use in practices, hospitals and nursing facilities in case of an epidemic.

Dosing instructions

For surface disinfection (wiping), produce a 3% working solution (15% RHEOSEPT-FD ultra plus and 85% water) and

allow to react for 4 hours. For instrument disinfection in acc. with RKI (Robert Koch Institut) list, efficacy are AB, produce a 6% working solution (30% RHEOSEPT-FD ultra plus and 70% water) and allow to react for at least 1 hour. For linen disinfection, produce 1.5% working solution (7.5% RHEOSEPT-FD ultra plus and 92.5% water) and allow to react for 12 hours. After disinfection wipe surface with clear water. Afterwards, we recommend to perform the treatment provided in the hygiene plan. An officially ordered decontamination is subject to mandatory documentation. Do not inhale fumes and ensure good ventilation of the area.

Contact times

Dosing specifications	Dosing details	Remark
RKI – laundry disinfection	1,5%	RKI list AB 12 hours
RKI – instrument disinfection	6%	RKI list AB 1 hour
RKI – surface disinfection	3%	RKI list AB 4 hours

Composition

Formalin 20% (DAB) (German Pharmacopoeia)

Physico-chemical data

- > pH-value (concentrate) 3,5 - 4,5

Labelling in accordance with the EC-directives

- > H341 - Suspected of causing genetic defects.
- > H301 + H311 + H331 - Toxic if swallowed, in contact with skin or if inhaled.
- > H314 - Causes severe skin burns and eye damage.
- > H317 - May cause an allergic skin reaction.
- > H350 - May cause cancer.
- > H370 - Causes damage to organs.
- > H335 - May cause respiratory irritation.
- > P280 - Wear protective gloves/protective clothing/eye protection/face protection.
- > P301 + P310 - IF SWALLOWED: Immediately call a POISON CENTER/doctor.
- > P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- > P271 - Use only outdoors or in a well-ventilated area.

Special notes

Please follow the safety instructions! Wear protective clothing! Wear hand and eye protection. Always store closed. Only for professional users. For industrial use.

Form of delivery

Article ID	Package size	Packaging unit
20505352	RHEOSEPT-D+ ultra bottle 1000 ml	carton
20505050	RHEOSEPT-D+ ultra canister 5 litres	canister

Accessories

48017002	Canister tap for 5 and 10-litres-canister with plug 51	piece
48038005	Canister key plug 51	piece
48020002	Canister pump plug 51mm, 30 ml pump stroke canister pump for 5 and 10-litres-canister blue or white with UN-code Riser pipe length 300 mm	piece
48029002	Bottle holder for 4 bottles à 1000 ml	piece
48041001	Protective glasses with integrated side protection and plastic glasses in acc. to EN 166 single-packed in bags, carton with 100 pieces	piece
48029006	Bottle holder for 6 bottles a 1000 ml stainless steel	piece
48036001	Protective gloves yellow, size S, package with 12 pairs 100 % natural latex, lined with cotton, rolled edging, suitable for the food processing industry	package
48036006	Protective gloves yellow, size M, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036005	Protective gloves yellow, size L, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036003	Protective gloves yellow, size XL, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036002	Protective gloves blue, 1 pair Latex, size 10, acid and alkaline-resistant (30 cm long) package with 12 pairs	package

Registrations

Listed with RKI in accordance with the rules §18 IfSG for governmental commended decontamination. Applicable in accordance with § 18 IfSG (infection protection law) and RKI-disinfection list (Robert Koch Institut). Equates to DAB (German Pharmacopoeia).

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information

RHEOSEPT®

about expert reports, please contact our office staff:
Telephone: +49 2241 - 3923-70 e-mail: info@rheosol.de.

Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.

RHEOSOL®

Wachendorff-Chemie GmbH • Langbaughstr. 15 • DE 53842 Troisdorf
☎ +49 2241 3923-0 📠 +49 2241 3923-90 ✉ info@rheosol.de 🌐 <http://www.rheosol.com>