

Product information: 20529

fast-acting disinfection

- aldehyde-free with fresh lemon scent
- extremely fast bactericidal and viricidal activity in case of TBC
- fast residue-free drying
- ready to use and alcoholic fast-acting disinfection



CE
0481

RHEOSEPT-SD plus citro

Alcohol-based fast-acting disinfectant

Product description

Aldehyde-free fast-acting disinfectant with fresh lemon scent for fast and thorough disinfection activities. The product dries with almost no residue. Spectrum of activity: bactericide incl. TBC, fungicide, viricide (HIV, HBV, HCV, noro, vaccinia, BVDV). This medical product is CE certified (CE 0481).

Fields of application

For prevention and control of infectious diseases. For the institutional use in surgeries, hospitals and maintenance institutions. For fast-acting disinfection of all water and alcohol resistant surfaces of medical products (not acryl or plexiglass).

Dosing instructions

Apply the liquid and ready to use solution on a cloth and wipe the dry surface evenly. Use a direct spray procedure

only when not inevitable. Depending on surface structure 20 to 50 ml per m². The residence time is to last minimum 1 minute. Do not use on acrylic (plexiglass). Proteins decrease the disinfection effect considerably. Product dries with almost no residue.

Spectrum of activity

Bactericide incl. TBC, fungicide, viricide (HIV/ HBV/ HCV/ Noro/ Vaccinia/ BVDV).

Contact times

Dosing specifications	Dosing details	Remark
contact time 60 seconds	undiluted	enveloped viruses (HIV, HBV, HCV, BVDV, vaccinia, influenza) EN 14476
contact time 5 minutes	undiluted	bactericide/fungicide (VAH) low load/mechanics
contact time 60 seconds	undiluted	bactericide/fungicide EN 1276, EN 13697, EN 1650
contact time 60 seconds	undiluted	mycobacteria (TBC)
contact time 60 seconds	undiluted	norovirus (MNV) EN 14476

Composition

100g contain: 37g ethanol, 13g propan-2-ol, 0.08g alkylbiguanide.

Physico-chemical data

- Flash point >21°C

Labelling in accordance with the EC-directives

- H319 - Causes serious eye irritation.
- H226 - Flammable liquid and vapour.
- H412 - Harmful to aquatic life with long lasting effects.
- P210 - Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P280 - Wear protective gloves/protective clothing/eye protection/face protection.
- P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P337 + P313 - If eye irritation persists: Get medical advice/attention.
- P403 + P235 - Store in a well-ventilated place. Keep cool.
- EUH208 - Contains polyhexamethylene biguanide hydrochloride. May produce an allergic reaction.

Special notes

A rinsing with water afterwards is generally not necessary. The product dries with almost no residue. Depending on the application area of the medical product the surface is to rinse carefully with water after the disinfection in order to remove possible product residues. Only for professional users. Note batch number and expiry date separately printed on the packaging. Expiry date: 36 months as of manufacturing date.

Forms of delivery

Product code	Package size	Packaging unit
20529352	RHEOSEPT-SD plus citro bottle 1000 ml	carton
20529050	RHEOSEPT-SD plus citro canister 5 litres	canister

Accessories

Product code	Description	Packaging unit
48006003	Sprayer Maxi bulk	piece
48020002	Canister pump plug 51mm, 30 ml pump stroke canister pump for 5 and 10-litres-canister blue or white with UN-code Riser pipe length 300 mm	piece
48038005	Canister key plug 51	piece
48017002	Canister tap for 5 and 10-litres-canister with plug 51	piece
48029002	Bottle holder for 4 bottles à 1000 ml	piece
48029006	Bottle holder for 6 bottles a 1000 ml stainless steel	piece
48041001	Protective glasses with integrated side protection and plastic glasses in acc. to EN 166 single-packed in bags, carton with 100 pieces	piece
48036001	Protective gloves yellow, size S, package with 12 pairs 100 % natural latex, lined with cotton, rolled edging, suitable for the food processing industry	package
48036006	Protective gloves yellow, size M, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036005	Protective gloves yellow, size L, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036003	Protective gloves yellow, size XL, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036002	Protective gloves blue, 1 pair Latex, size 10, acid and alkaline-resistant (30 cm long) package with 12 pairs	package

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information about expert reports, please contact our office staff:
Telephone: +49 2241 - 3923-70 e-mail: info@rheosol.de.

Admission

This medical product is CE certified (CE 0481).

Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.