

Product information: 20536

fast-acting disinfection

- ready to use and alcoholic disinfection wipes
- extremely fast bactericidal and viricidal activity in case of TBC
- aldehyde-free, without covering perfume
- fast residue-free drying
- skin-friendly - was awarded highest marks when dermatologically tested
- VAH listed



CE  
0481



## RHEOSEPT-SD plus wipes

Wipes with alcohol-based fast-acting disinfection

### Product description

Aldehyde-free wiping disinfection for fast-acting and extensive disinfection measures with pleasant scents. Wet disinfection wipes in size 130x200 mm, dipped in RHEOSEPT SD plus. The product dries with almost no residue. Spectrum of activity: bactericide incl. TBC, fungicide, viricide (HIV, HBV, HCV, noro, vaccinia, BVDV). This medical product is CE certified (CE 0481).

### Fields of application

For fast-acting disinfection of all water and alcohol resistant surfaces of medical products (not acryl or plexiglass). For the institutional use in surgeries, hospitals and maintenance institutions.

### Dosing instructions

Detach surface from contamination and wetness before disinfection. Wipe the dry surface evenly with the wipes. The

surface is to wet sufficiently. The residence time is to last minimum 1 minute. Do not use on acrylic (plexiglass). Proteins decrease the disinfection effect considerably.

### Spectrum of activity

Bactericide incl. TBC, fungicide, viricide (HIV/ HBV/ HCV/ Noro/ Vaccinia/ BVDV).

## Contact times

Dosing specifications	Dosing details	Remark
contact time 60 seconds	undiluted	enveloped viruses (HIV, HBV, HCV, BVDV, vaccinia, influenza) EN 14476
contact time 60 seconds	undiluted	norovirus (MNV) EN 14476
contact time 60 seconds	undiluted	bactericide/fungicide EN 1276, EN 13697, EN 1650
contact time 5 minutes	undiluted	bactericide/fungicide (VAH) low load/mechanics
contact time 60 seconds	undiluted	mycobacteria (TBC)

## Composition

100g contain 37g ethanol, 13g propanol, 0.08g alkylbiguanid.

## Physico-chemical data

- Flash point <21°C

## Labelling in accordance with the EC-directives

- H226 - Flammable liquid and vapour.
- H319 - Causes serious eye irritation.
- H412 - Harmful to aquatic life with long lasting effects.
- P210 - Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P280 - Wear protective gloves/protective clothing/eye protection/face protection.
- P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P337 + P313 - If eye irritation persists: Get medical advice/attention.
- P403 + P235 - Store in a well-ventilated place. Keep cool.
- EUH208 - Contains polyhexamethylene biguanide hydrochloride. May produce an allergic reaction.

## Special notes

A rinsing with water afterwards is generally not necessary. The product dries with almost no residue. Depending on the application area of the medical product the surface is to rinse carefully with water after the disinfection in order to remove possible product residues. Note batch number and expiry date separately printed on the packaging. Only for professional users.

## Forms of delivery

Product code	Package size	Packaging unit
20536001	RHEOSEPT-SD plus wipes dispenser box 100 wet wipes 130x200 mm	carton

## Certificate and information

For additional information on this product, please see → [www.rheosol.com](http://www.rheosol.com) – For specific questions or information about expert reports, please contact our office staff: Telephone: +49 2241 - 3923-70 e-mail: [info@rheosol.de](mailto:info@rheosol.de).

## Zulassung

This medical product is CE certified (CE 0481).

## Environmental information

Wachendorff-Chemie GmbH produce under application of advanced, safe and environmentally friendly procedures and in compliance with high quality standards.