

Product information: 20568

treatment of aspiration system

- highly effective hygienic preparation of suction units and amalgam separators
- dissolve stubborn dirt and bind rest of salivary and drilling dust
- hygienic preservation of collecting repository
- high material compatibility
- free of aldehyd, phenol and chlorine



CE
0481

RHEOSEPT-Suction treatment D

Disinfectant concentrate for suction systems in dental practices

Product description

Low-foaming cleaning concentrate for suction systems and amalgam separators for daily treatment. Hygienic through preserving and microbiocidal effects. High cleaning power with respect to deposits, saliva residues and drilling dust. This medical product is CE-certified.

Fields of application

RHEOSEPT-ID dental suction system disinfectant is a concentrate for the disinfection and cleaning of all types of dental suction systems and amalgam separators in the medical area.

Dosing instructions

Allow 2% solution (up to max. 60°C) to react for 15 minutes and suck off immediately. Clean system according to the manufacturers of the devices. This product is biodegradable and can be safely disposed of via the waste water system.

We recommend to use RHEOSEPT-Suction treatment W as weekly interval cleaning to remove sediments.

Contact times

Dosing specifications	Dosing details	Remark
15 minutes	2%	at max. 60°C of cleaning temperature

Composition

5-10% cationic tenside, 5-10% non-ionic tenside. Further ingredients: alcohol, complexing agent, dye.

Physico-chemical data

- pH-value (concentrate) 12-12.5

Labelling in accordance with the EC-directives

- H314 - Causes severe skin burns and eye damage.
- H332 - Harmful if inhaled.
- P260 - Do not breathe dust/fume/gas/mist/vapours/spray.
- P305 + P351 + P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- EUH401 - To avoid risks to human health and the environment, comply with the instructions for use.

Forms of delivery

Article ID	Package size	Packaging unit
20568050	RHEOSEPT-Suction treatment D canister 5 litres	canister

Accessories

48017002	Canister tap for 5 and 10-litres-canister with plug 51	piece
48020002	Canister pump plug 51mm, 30 ml pump stroke canister pump for 5 and 10-litres-canister blue or white with UN-code Riser pipe length 300 mm	piece
48038005	Canister key plug 51	piece
48041001	Protective glasses with integrated side protection and plastic glasses in acc. to EN 166 single-packed in bags, carton with 100 pieces	piece
48036001	Protective gloves yellow, size S, package with 12 pairs 100 % natural latex, lined with cotton, rolled edging, suitable for the food processing industry	package
48036006	Protective gloves yellow, size M, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036005	Protective gloves yellow, size L, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036003	Protective gloves yellow, size XL, package with 12 pairs 100 % natural latex, lined cotton, rolled edging, suitable for the food processing industry	package
48036002	Protective gloves blue, 1 pair Latex, size 10, acid and alkaline-resistant (30 cm long) package with 12 pairs	package

Specific advices

Store closed, frost-protected and cool.

Registrations

This medical product is CE-certified.

Certificate and information

For additional information on this product, please see → www.rheosol.com – For specific questions or information about expert reports, please contact our office staff: Telephone: +49 2241 - 3923-70 e-mail: info@rheosol.de.

Environmental information

Wachendorff-Chemie GmbH produce under application of

RHEOSEPT®

advanced, safe and environmentally friendly procedures and
in compliance with high quality standards.

RHEOSOL®

Wachendorff-Chemie GmbH • Langbaughstr. 15 • DE 53842 Troisdorf
☎ +49 2241 3923-0 📠 +49 2241 3923-90 ✉ info@rheosol.de 🌐 <http://www.rheosol.com>